## RACE® EVO

## **6% SEQUENCE**









RE1 - 15/.04

(7, 800-1000 (1.5 Ncm)



- Create straight-line coronal and radicular access
- Use an ISO 010 hand file to explore the canal
- Determine the working length (WL)



- Use RACE EVO files with gentle 2-3 mm strokes, applying very light apical pressure and allowing the file to progress passively along the canal. After 3 strokes, remove and clean the file, then irrigate; recapitulate
- Use **RE1** to perform glide path until the WL is reached
- Start the shaping procedure by using **RE2** until the WL is reached
- Use **RE3** for the shaping and final apical preparation If necessary, use larger sizes for an apical enlargement



- Once the canal has been shaped, proceed with the cleaning of the canal
- Obturate by selecting the appropriate gutta-percha point and sealer













RE1


INITIAL **SHAPING** 







Standard canal (most cases)

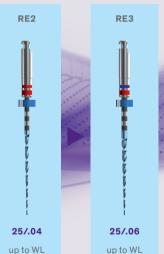

Medium canal


30/.06

up to WL



Large

canal

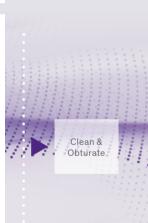





15/.04

up to WL